
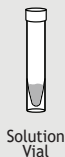


ASSURE

SURVEILLANCE S-PACK

For Surveillance Use Only | Not EUA approved for diagnostic or screening use | Use in a pooled test setting to de-identify results

KIT CONTENTS

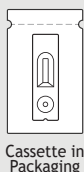

TEST PROCEDURE STEPS

- 1** PLAN how many people you want to test. The S-Pack can test between 2 and 4 people. **DO NOT** use for only 1 person.




- 2** Give each person being tested 1 nasal swab for their own use. **REMOVE** the swab from the packaging. Do not touch the swab tip.



- 3** For each person, **INSERT** entire soft part of the swab into left nostril (1/2-3/4 inches). Rub swab against the inside wall of nostril. Make at least 5 big circles, taking a total of 15 sec. **Do not just spin the swab.** Repeat with right nostril.



- 4** After swabbing each person, **PLACE** any swab into the vial one swab at a time, tip first. **GENTLY** plunge the swab up and down in the vial 5 times.

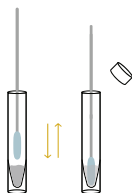

- 5** After plunging the first swab, discard and plunge the next swab, **REPEATING STEP 4**. The same vial will be used for all people.

Pro tip:

If you test only 3 people, use 3 different swabs and do STEP 4 for each swab.


If you test only 2 people, use 2 swabs and do STEP 4 for each swab.

- 6** Place test cassette flat on a level surface and **POUR** liquid content from vial gently into the cassette well. It is normal for some of the liquid to remain in the bottom of the vial.



- 7** **WAIT** 20 minutes. Results interpretation continued on other side.

20 min

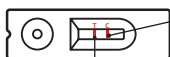

Control Line

COVID-19 Line

See Back




For Surveillance Use Only | Not EUA approved for diagnostic or screening use | Use in a pooled test setting to de-identify results

RESULT INTERPRETATION

NOTES:

- Read the results at 20 minutes
- Do not read the results before 20 minutes or after 30 minutes

Positive Results may appear as bold or faint as shown in the examples below.

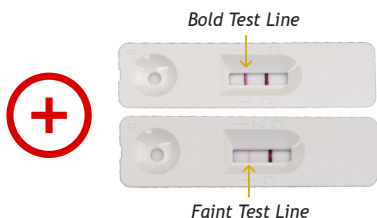

Negative results will only show a control line.



Invalid results when no control line is present.

