
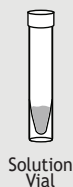



ASSURE-100

KIT CONTENTS

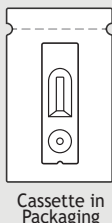


TEST PROCEDURE STEPS


- 1** REMOVE nasal swab from packaging. Do not touch the swab tip.

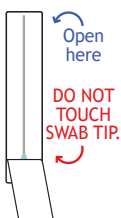


- 2** INSERT entire soft part of the swab into one nostril ($\frac{1}{2}$ - $\frac{3}{4}$ inches). Rub swab against the inside wall of nostril. Make at least 5 big circles, taking a total of 15 seconds. Do not just spin the swab. Repeat in the other nostril.

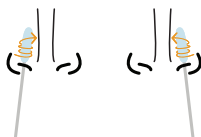


- 3** After swabbing each nostril, PLACE swab into the vial, tip first. PLUNGE swab in vial for 10 seconds by rapidly moving swab up and down.

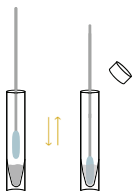


- 4** PUSH swab all the way to the bottom of the vial and leave for one minute. Following this, discard swab and open cassette package.

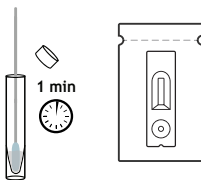

- 5** Place test cassette flat on a level surface and POUR liquid content gently into the cassette well. It is normal for some of the liquid to remain in the bottom of the vial.

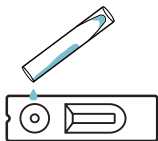

- 6** WAIT 20 minutes. Results interpretation continued on other side.

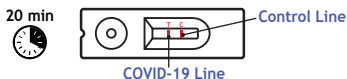

See Back 




ASSURE support line: Further information can be obtained by contacting info@assure-test.com or by calling +1.808.500.7574

RESULT INTERPRETATION

NOTES:

- Read the results at 20 minutes
- Do not read the results before 20 minutes or after 30 minutes

Positive Results may appear as bold or faint as shown in the examples below.

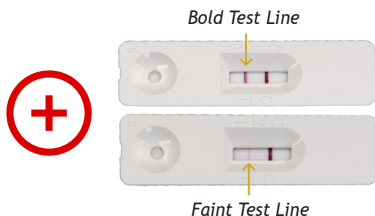

Negative results will only show a control line.



Invalid results when no control line is present.

