


# ASSURE

— SURVEILLANCE —

For Surveillance Use Only | Not EUA approved for diagnostic or screening use | Use in a pooled test setting to de-identify results

## KIT CONTENTS

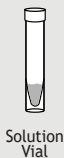


## TEST PROCEDURE STEPS


- 1** REMOVE nasal swab from packaging. Do not touch the swab tip.




- 2** INSERT entire soft part of the swab into left nostril ( $\frac{1}{2}$ - $\frac{3}{4}$  inches). Rub swab against the inside wall of nostril. Make at least 5 big circles, taking a total of 15 seconds. Do not just spin the swab. Repeat with right nostril.

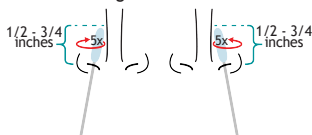


- 3** After swabbing each nostril, PLACE swab into the vial, tip first. PLUNGE swab in vial for 10 seconds by rapidly moving swab up and down.

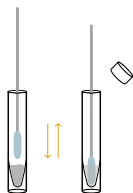

- 4** PUSH swab all the way to the bottom of the vial and leave for one minute. Following this, discard swab and open cassette package.

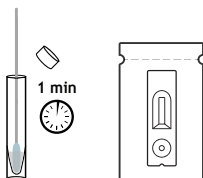

- 5** Place test cassette flat on a level surface and POUR liquid content gently into the cassette well. It is normal for some of the liquid to remain in the bottom of the vial.

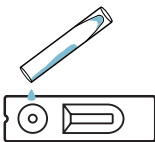

- 6** Wait until a red or pink color appears in cassette window, then place cassette into box to de-identify results. If color does not appear in cassette window, add additional liquid to cassette well.

